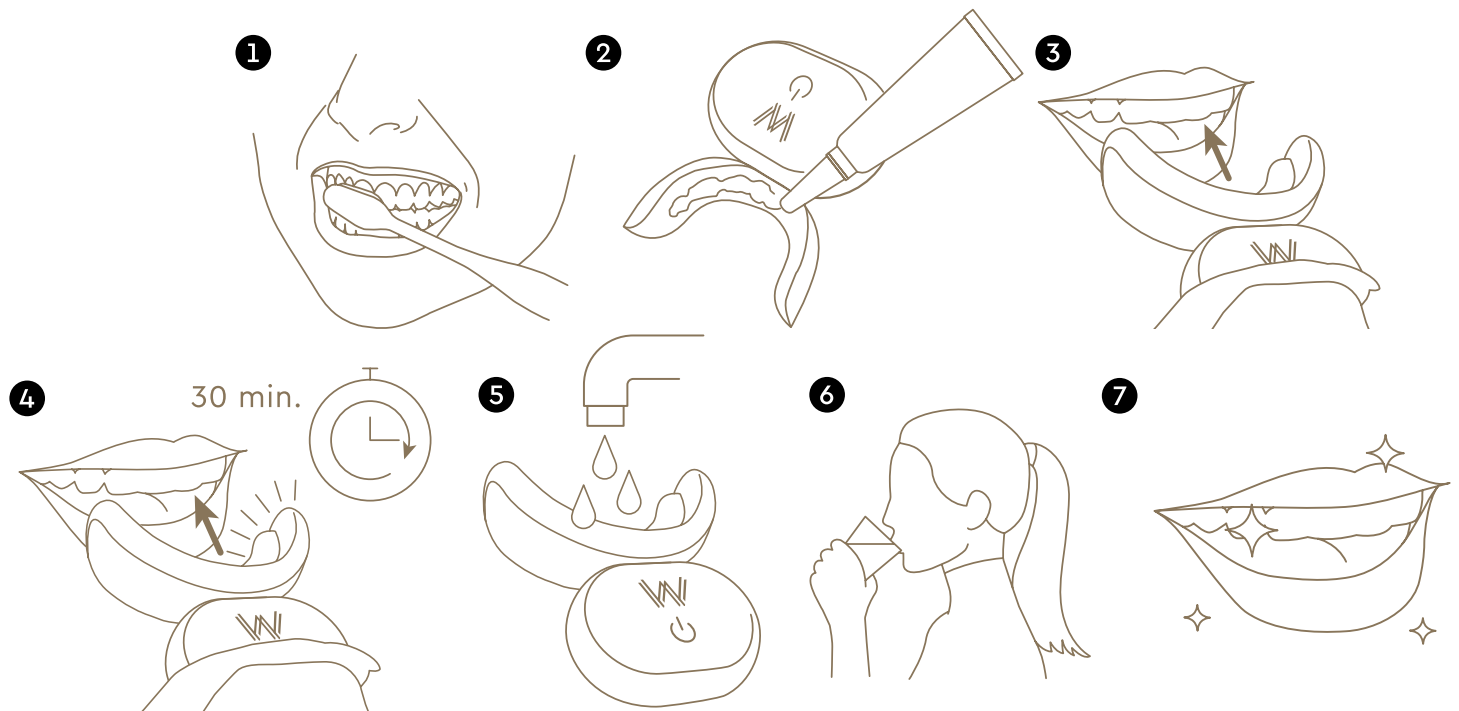
WHITE ROYALE PEROXIDE FREE TEETH WHITENING

1. Brush your teeth thoroughly.
2. Apply a generous amount of gel to the inside of the mouth tray, focusing on the areas that will be in direct contact with your teeth. Ensure the gel is mainly positioned at tooth level.
3. Place the tray in your mouth and ensure a proper fit.
4. Activate the built-in LED of the tray.
 - a. All color modes function identically.
 - b. The light switches off automatically after 15 minutes.
 - c. Reactivate the light and keep the tray in for a total of 30 minutes.
5. Remove the tray after 30 minutes and rinse it with lukewarm water.
6. Rinse your mouth thoroughly with lukewarm water.
7. Check the result and enjoy a fresher, brighter smile.

DESCRIPTION

Cavex Peroxide Free Teeth Whitening is a powerful yet gentle treatment that delivers a visibly whiter, more radiant smile in just a few applications. The peroxide-free gel whitens teeth without causing sensitivity and helps prevent new discoloration.

INDICATIONS FOR USE

White Royale Peroxide Free Teeth Whitening is a whitening product intended to lighten the color of natural teeth. Suitable for individuals aged 18 and over. A consultation with a qualified dentist is recommended prior to treatment to check for any underlying oral or health issues.

WARNINGS

Avoid contact with eyes. Keep out of reach of children. Do not use on hypersensitive teeth.

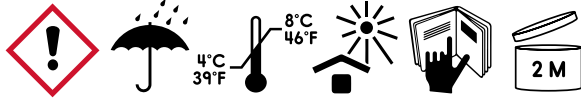
STORAGE INSTRUCTIONS

Store in a dark, dry place. Avoid exposure to direct sunlight or high temperatures. Keep refrigerated when not in use. Limited shelf life after opening.

WHITE ROYALE PEROXIDE FREE TEETH WHITENING


INGREDIENTS

Glycerin, Water (Aqua), Phthalimidoperoxycaproic Acid (PAP), Sorbitol, Erythritol, Ammonium Acryloyldimethyltaurate/VP Copolymer, Sodium Polyphosphate, PVP, Menthol, Eucalyptol, Potassium Hydroxide.

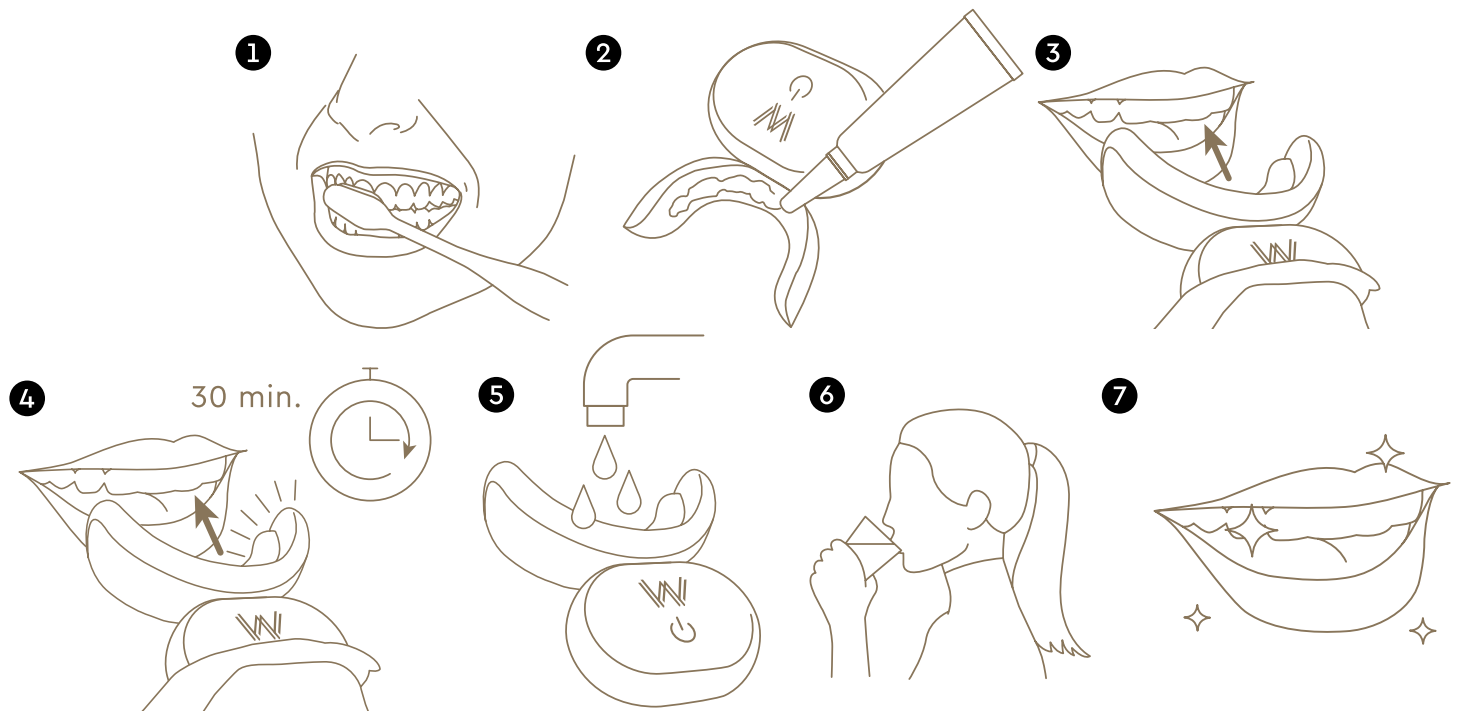 Cavex Holland BV
Fustweg 5
2031CJ Haarlem
The Netherlands
T +31 23 5307700
dental@cavex.nl
cavex.nl

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